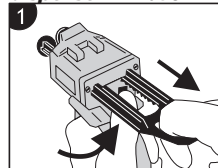
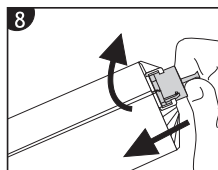
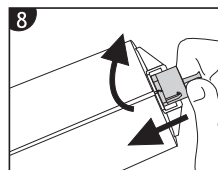
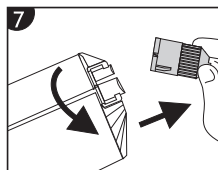
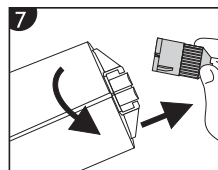
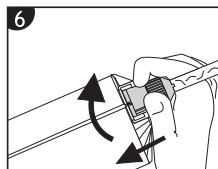
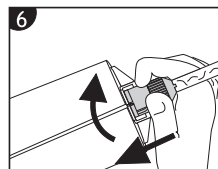
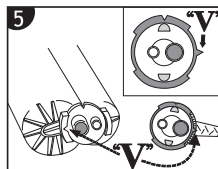
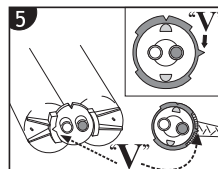
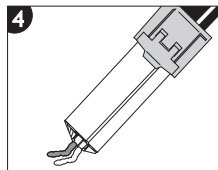
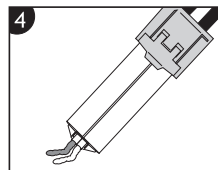
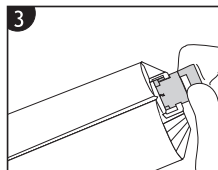
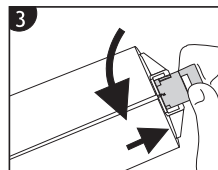
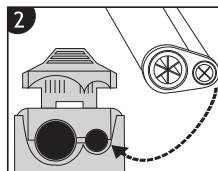
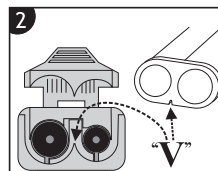
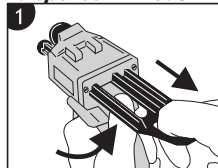
dispenser D2 ratio 1:1**dispenser D2 ratio 4:1****DISPENSERN****(SV)**

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DISPENSER **EN**

- Pull the dispenser rack as far back as it will go by raising the black lever below it. Lift the cartridge lock. (.1)
- Insert the cartridge making sure that it goes in the correct way round. For the dispenser rack 1:1 model, the cartridge must be inserted with the "V" shape on its base turned downwards. The cartridge lock must then go down easily (.2)
- When the cartridge is in position on the dispenser, remove the cap by turning it 1/4 turn. Remove the cap in a clean motion. Keep the cap (.3)
- Check that the two components come out together when you press on the dispenser lever to let a small amount of product out. Clean the dispenser, removing product from the holes through which it comes out (.4)
- Insert the mixing tip in a clean motion so that the V-shaped notch on the coloured ring is aligned with its housing in the cartridge. Turn 1/4 turn, exerting pressure only on the coloured ring and not on the mixing tip (.5-6)
- The device is now ready for use by pressing lightly on the dispenser lever.
- Remove the mixing tip after use and put the cap back in place in the way described above (.7-8)
- Follow the instructions from the beginning when you need to replace the cartridge.

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