VAP 8 / 8A

- con pedale
- with foot pedal
- avec pédale
- con pedal
- mit Pedal
- с педалью

USER MANUAL



Steam cleaners &
water softener

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


Zhermack
Dental

A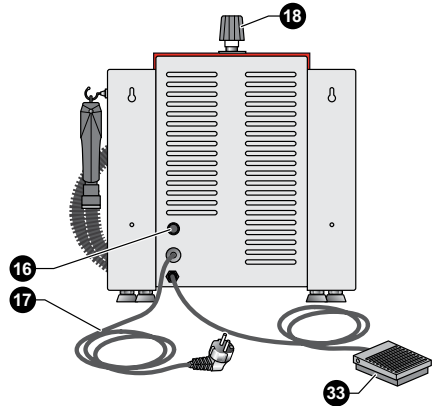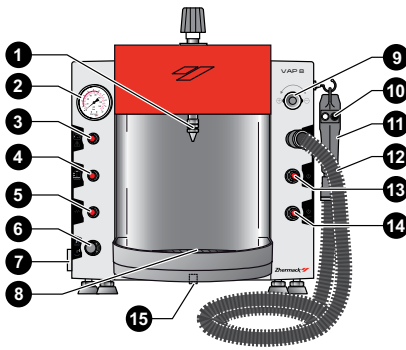

VAP 8 - con pedale - con pedal
 - with foot pedal - mit Pedal
 - avec pédale - с педалью

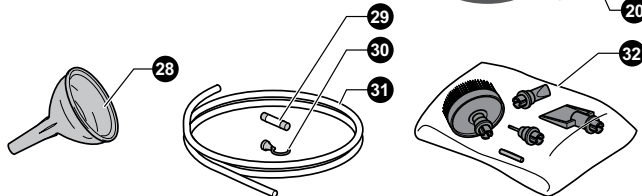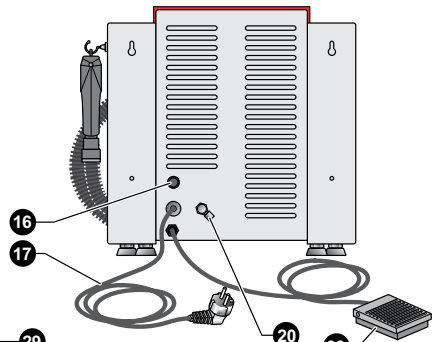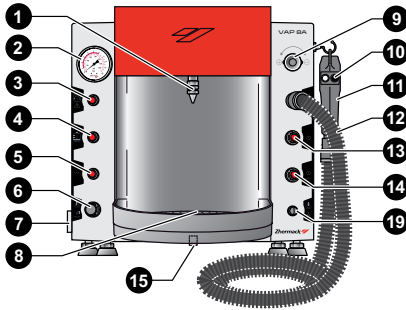VAP 8A - con pedale - con pedal
 - with foot pedal - mit Pedal
 - avec pédale - с педалью

C

Zhermack    

A Zhermack S.p.A. - via Bovazecchino, 100
 45021 Badia Polesine (RO) Italy - Tel. +39 0425 597 611

B C305227 - VAP 8 C/PED
 SN:20000000

D 230V~50/60Hz 1700W **C**

Zhermack    

A Zhermack S.p.A. - via Bovazecchino, 100
 45021 Badia Polesine (RO) Italy - Tel. +39 0425 597 611

B C305226 - VAP 8A C/PED
 SN:20000000

D 230V~50/60Hz 1700W **C**

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3.2 DESCRIPTION OF THE COMPONENTS VAP 8 with foot pedal / VAP 8A with foot pedal - Fig. A

1. Fixed nozzle
2. Pressure gauge
3. Overpressure warning light
4. Water LOW warning light
5. Element ON indicator light
6. Fixed nozzle button
7. Metallic disk
8. Water tray
9. Steam flow adjuster knob
10. Gun trigger
11. Gun
12. Gun connection lead
13. Main switch
14. Element switch
15. Water drain plug
16. Fuse
17. Power lead
18. Water filler plug (VAP 8)
19. Water/steam mode selector (VAP 8A)
20. Water filler connection (VAP 8A)
33. Foot control

3.4 IDENTIFICATION DATA - Fig. C

- A Name and address of the Manufacturer
- B Equipment code and model
- C Serial number
- D Power supply information

7.3 START-UP AND OPERATION

To deliver steam from the fixed nozzle, place the object being processed under the nozzle (1) and press the fixed nozzle button (6) or press the pedal (33).

- When using the fixed nozzle, the operator should touch the special metallic disk (7, Fig. A) located on the side of the appliance, so as to discharge any static electricity.

CHAPTER 10: ATTACHMENTS

10.1 EC DECLARATION OF CONFORMITY

The Manufacturer:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597611 • Fax +39 0425 53596

Person authorised to compile the technical file:

Zhermack S.p.A.
Via Bovazecchino, 100
45021 Badia Polesine • RO • Italy
Tel. +39 0425 597611 • Fax +39 0425 53596

declares under its sole responsibility that the product:

Vap 8 with foot pedal
Vap 8A with foot pedal

is in conformity with Directive 2011/65 EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

It is also in conformity with the laws implementing the following directives:

Directive 2004/108 EC (EMC Directive) and subsequent amendments.
Directive 2006/95 EC (Low Voltage Directive) and subsequent amendments.
Directive 97/23 EC (Pressure Equipment).

Please note that the manufacturer's liability for damages caused by a defective product expires 10 years from the date on which the manufacturer released such product on the market and that such liability is transferred to the user in accordance with Directive 85/374/EEC, subsequent amendments and the laws implementing it.

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